A

A

DEUTSCH

ENGLISH

FRANÇAIS

ESPAÑOL

PORTUGUES

ITALIANO

NEDERLANDS

SVENSKA

Monopolare Pinzetten
Monopolar Forces
Pinzas monopolares
Pinças monopolares
Pinze monopolaris
Monopolaire pincetten
Monopolaire pincerter

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Non-sterile. - Clean and sterilize before first and subsequent uses.

Electrosurgical accessories may only be used and disposed of by trained medical staff! These instructions do not substitute the instructions for the electrosurgical unit and other accessories used.

Intended use:
 Monopolar coagulation of soft tissue and tubular structures.

Prior to use:
 Examine the integrity of the insulation as well as the cleanliness and integrity of the forceps.
 △ Only use safe and sterilized products!
 Connect forceps properly to the correct cable.
 △ Connect forceps and cable only to an electrosurgical unit that has been switched off or is in standby mode.
 Nicht beachten kann zu Verbrennungen und elektrischen Schlägen führen!

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Cleaning and sterilization:

Observe nationally applicable regulations and guidelines.
 Disconnect cable from forceps.
 Do not allow blood and debris to dry on forceps.
 Ne laissez pas les résidus de sang et de tissus se dessécher!
 Manual cleaning:
 Use a soft cloth or brush to remove blood and debris.
 => Do not use aggressive/abrasive cleaners. Rinse thoroughly with clean tap water after cleaning.
 Do not bend the branches of the forceps apart. (FIG1)
 Machine cleaning/disinfection:
 Use an approved cleaning programme (>10 min at 93 °C). We recommend enzymatic cleaner. Follow the manufacturer's dosage instructions. Cleaning in ultrasound possible. Ultrasound may shorten the life of the instrument.
 => Perform visual control prior to sterilization and verify the integrity of the insulation as well as cleanliness and integrity of the entire instrument.
 Only sterilize clean/disinfected instruments. Use paper/foil wrapping or appropriate sterilization containers.
 Sterilization in the autoclave (fractioned pre-vacuum):
 121 °C (250 °F), 20 min or 134 °C (273 °F), 5 min.
 Max. temperature 138 °C, max. sterilization time 18 min.
 => Sterilization at high temperatures and long sterilization times shorten the life cycle of the instrument.
 Do not sterilize in hot air.
 Do not sterilize in STERRAD®.

△ Sutter cannot be held liable for changes to the product after purchase or for deviations from these instructions.

Information on applicable standards, regulations and existing validations may be obtained from the manufacturer.

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© Sutter
STERRAD® is a trademark of Johnson & Johnson, Inc.