Sterilisationscontainer Sterilization Containers

REF 800002

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English

1 Product/User/Disposal

Medical devices may only be used and disposed of by trained medical staff!

2 Intended use

Sterilization containers are intended to hold reusable instruments for sterilization and storage purposes.

Sterilization containers require suitable disposable filters TAB1[A]. These have to be exchanged prior to each sterilization.

1.1 Recommended sterilization procedure

The sterilization containers have been validated according to the following parameters:

3 x pre-vacuum steam sterilization; 134 ° C (273 ° F)

3 minutes hold time; 10 minutes drying time

Loading: Medical instruments (e. g. surgical instruments such as forceps, scissors, clamps).

3 Handling

3.1 General

The sterilization containers are made of an aluminum alloy with an anodized surface.

Ensure that you strictly observe the information for suitable cleaning agents.

The latch may be secured with a tamper-evident seal TAB1[B] that cannot be opened without being destroyed.

Only an intact seal can guarantee that the container has not been opened by anyone who is not authorized to do so.

2.2 Preparations for cleaning

1. Remove lid from bottom part.

2. Remove contents of container (e.g. storage tray, instruments etc.)

3. Dislodge the filter retention plates from the inside of the lid and the container bottom.

4. Discard single-use filters.

5. Remove disposable seal and identification tags.

Important: Paper filters are for single-use only and must be discarded and replaced after use.

3.3 First use of a new container

Clean thoroughly prior to first use.

After cleaning the container, insert a new filter (see 3.6 filter change).

3.4 Cleaning and disinfection

Use mild cleaners with a neutral pH value (pH7). All cleaning agents must be free from sodium, alkali, and carbonates. We recommend the use of fully desalinated water (quality standard according to EN 285 Annex B).

Clean and disinfect the container prior to first use.

Containers used for devices for disposal must be cleaned and disinfected after each use.

3.4.1 Manual cleaning

Use suitable cleaning agents according to 3.4. Use a sponge and water for cleaning.

After cleaning rinse the container carefully and dry manually.

Do not employ metal brushes or abrasive products.

3.4.2 Automated cleaning

Automated (machine) cleaning is recommended when the washer has a special cleaning program for aluminum containers.

Remove lid from container bottom part.

Place container bottom part with the opening downwards into the washer.

Place container lid with the inside downwards and locking devices folded inwards.

Dismantle the container into its components (bottom part, lid, filter holder) and load into specially designed washer baskets.

1 minute pre-cleaning with cold water.

3 minutes cleaning with Mediclean 0.5 % (Dr. Weigert)* at 45 °C (113 °F).

Neutralization with demineralized water.

*When employing another cleaning agent only use cleaning agents which have been officially approved for the sterilization of aluminum sterilization containers.

After termination of the cleaning program use a dry soft cloth or medical compressed air to dry the container and its parts.

Traces (adhesive labels, indicator strips, markings) that remain after any of the usual cleaning procedures may be removed with an anodized cleaning agent. Afterwards proceed to clean the containers as usual.

3.5 Function check

Prior to each use check if the sterilization containers function properly.

Do not use the sterilization containers if latches, seals, filter holders and filters are damaged or if parts of the container are bent out of shape or dented.

3.6 Filter change

After changing the filter push the filter holder back into the right position. When positioned correctly, it will engage audibly.

Single-use paper sterilization filters TAB1[A] must be exchanged prior to each sterilization procedure.

Important: Only reassemble the given parts such as lid, bottom part, filter, seals, and filter holders to retain the impermeability and germ barrier.

4 Container loading

The total weight of a container load may not exceed 1.8 kg. If higher, satisfactory sterilization cannot be guaranteed.

5 Placement in sterilizer

The containers have been designed for use in any of the commercially available large sterilizers with moist heat.

Important: Observe the manufacturer's instruction for the loading of the sterilizer. Do not use additional outer packaging for the container.

Do not cover the perforated sections of lid and bottom.

During loading, unloading and transport always lift the sterilization container by its handles and not the lid.

6 Process control

Load and run the sterilizer according to the instructions of the manufacturer of the sterilizer (temperature, sterilization time, etc.).

To reduce condensation as much as possible, leave the container on the cart until the container has cooled down to the point where it can be touched safely.

After each sterilization verify its effectiveness by using additional monitoring indicators, and ensure that the lid and bottom of the container (closed shutters) are firmly joined.

7 Storage

Storage time depends on the storage conditions. It must be determined by the responsible hygiene specialist.

Recommended storage conditions:

Temperature: 15 to 26 °C (59 °F to 79 °F)

Humidity: 30 to 50%

Air pressure: normal atmospheric pressure

Recommended storage time:

6 weeks in open racks

3 months under protected conditions (e. g. closed storage shelves)

For especially high asepsis demands shorter storage times or additional packaging are recommended.

8 Maintenance / Repair

The useful life of the seals is up to 500 sterilization cycles. After this time verify and exchange the seals if necessary. Exchange damaged seals promptly.

Do not treat seals with sprays, oils or solvents. For cleaning and care it is sufficient to wipe the seals periodically with a damp cloth.

Damage to a sterilization container must be repaired without delay.

Maintenance and repairs may only be performed by trained specialists.

9 Materials

The sterilization containers are made of an anodized aluminum alloy. Do not use metal brushes, steel wool, scratch sponges, and other abrasive cleaning devices.

10 Accessories

Sutter Medizintechnik cannot be held liable for changes to the product after purchase or for deviations from these instructions.

Subject to change. Updates available under www.sutter-med.de.

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12/2012 © Sutter Medizintechnik GmbH