

Calvian endo-pen® Bipolar Forceps

Please pass to Sterile Processing Department!



If necessary, use a soft cloth or brush.
Do not use a wire brush, scalpel, scratch sponge
or similar item.




This instrument shall not and can not be disassembled.



Do not immerse in hydrogen peroxide (H₂O₂).



To protect the instrument please use
the storage trays.

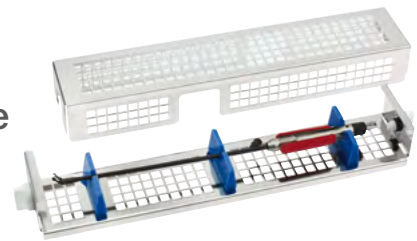

The lumina of the instruments must be connected
to the flushing connection of the CDD* using the
Luer-Lock connections provided.

*Cleaning and Disinfection Device



The overall reconditioning process encompasses pre-cleaning, cleaning/disinfection, and sterilization.
Please refer to the instrument's instructions for use for more information and observe national guidelines
and regulations.

© Sutter Medizintechnik GmbH · Subject to change · REF 1426_00 - U07



PRECISION ELECTROSURGERY
Made in Germany

SUTTER MEDIZINTECHNIK GMBH
ALFRED-WALZ-STR. 22 · 79312 EMMENDINGEN/GERMANY
TEL. +49(0)7641-96256-0 · FAX +49(0)7641-96256-30
WWW.SUTTER-MED.COM · INFO@SUTTER-MED.DE