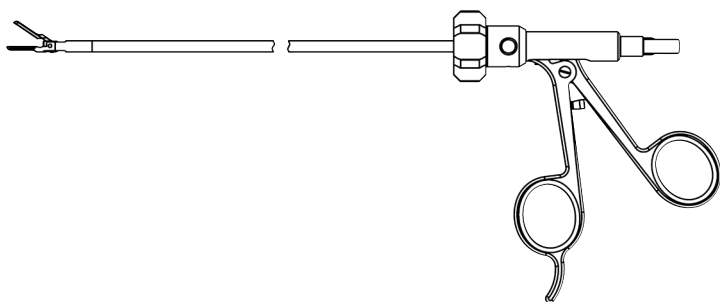
# Bison™

Bipolare Zange	DE
Bipolar Forceps	EN
Pince Bipolaire	FR
Pinzas Bipolares	ES
Pinza Bipolare	IT
Bipolaire Tang	NL

**REF:**

73 01 00 -00 – 73 01 99 -99

73 01 02 -33

73 02 00 -00 – 73 02 99 -99

73 02 02 -33

73 03 00 -33 – 73 03 99 -99

73 04 00 – 73 04 99

73 05 00

**CE** 0297

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Tullastr. 87

79108 Freiburg/Germany

Tel.: +49 (0) 761 515510

Fax: +49 (0) 761 5155130

E-Mail: [info@sutter-med.de](mailto:info@sutter-med.de)

[www.sutter-med.de](http://www.sutter-med.de)

TAB A	REF
TAB A <sub>1</sub> 	93 01 00

FIG A	REF
FIG A: 1-8	73 01 00-XX - 73 01 99-XX
FIG A: 7	73 02 00-XX - 73 02 99-XX
FIG A: 2a/2b	73 03 00-XX - 73 03 99-XX
FIG A: 8	73 04 00-XX - 73 04 99-XX
FIG A: 3-6	73 05 00

FIG. A

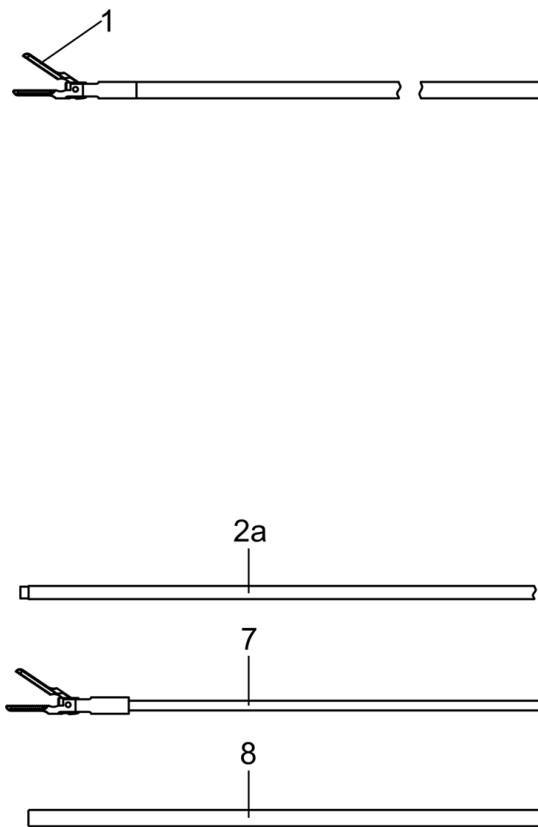
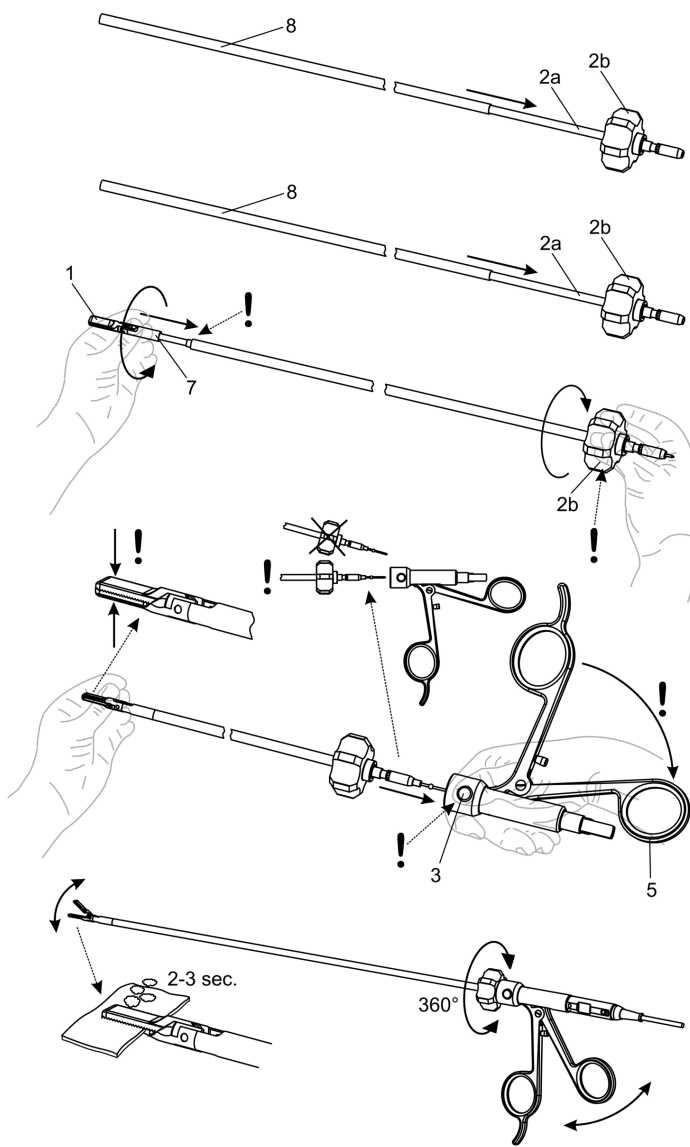
FIG. B

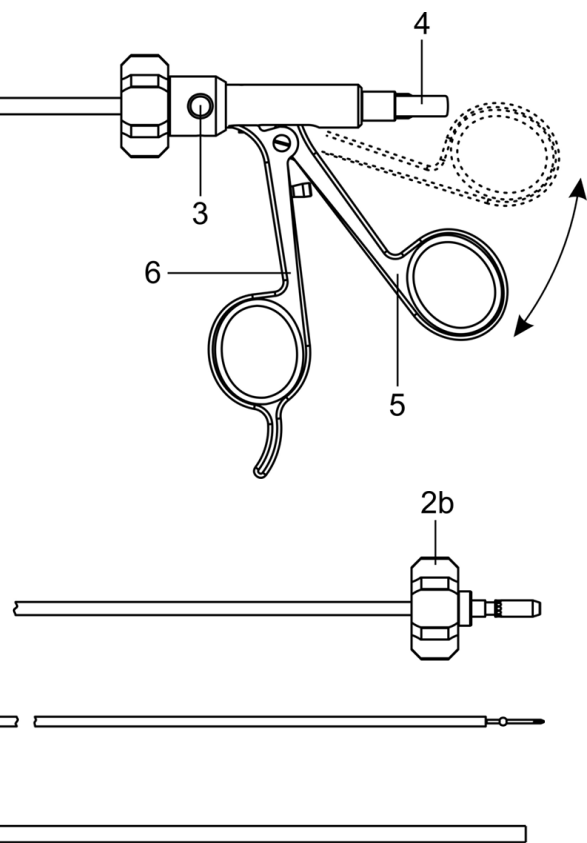
FIG. C

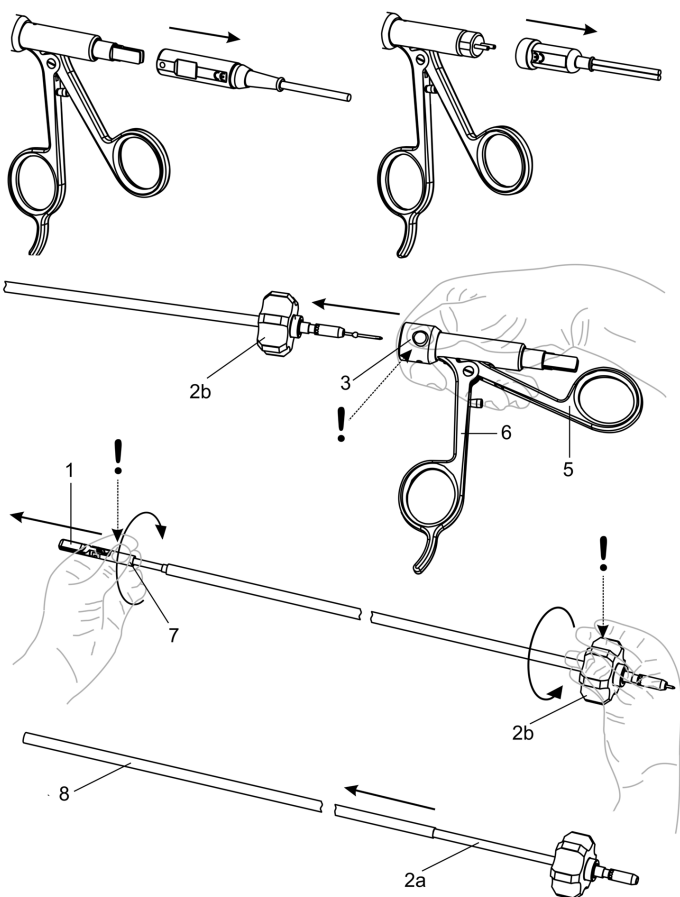
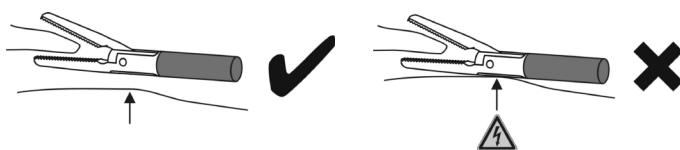
FIG. D

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STERRAD® is a trademark of Johnson & Johnson, Inc.

FIG A (Komponenten)

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FIG C (Demontage)

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**Product / User / Disposal:**

Electrosurgery accessories may only be used and disposed of by qualified medical personnel!

Notwithstanding these instructions, reading the instructions for use provided with the electrosurgery device and other accessories being used is mandatory.

⚠ **Not sterile.** Must be cleaned and sterilized prior to initial and each subsequent use.

**Intended use:**

Bipolar forceps for grasping, coagulating, and cutting soft tissue during minimally invasive surgical procedures.

**Service life:**

Up to 20 reconditioning cycles can be expected with proper use. This depends on the intensity of use and the applied parameters.

**Electrosurgery cables:**

The Sutter bipolar forceps are intended for use with bipolar silicone cables with US male connector or European flat connector from the manufacturer Sutter Medizintechnik GmbH.

**Components (FIG A):**

- 1 Moveable jaw part
- 2a Shaft
- 2b Rotary knob for 360° rotation
- 3 Push button
- 4 Plug
- 5 Moveable handle part
- 6 Rigid handle part
- 7 Electrode insert (including jaw part)
- 8 Insulating tube

**Before using:**

⚠ Inspect the product for cleanliness, mechanical function, and intact insulation before each use.

We recommend checking the insulation with a suitable test device.

⚠ Only use sterilized products that are in flawless condition!

A certain discoloration of the non-stick instrument tip is normal and harmless.

To connect the forceps and cables, the electrosurgery device must be turned off or in standby mode. Failure to comply may lead to burns and electric shocks!

For further information about electrical safety, we recommend DIN EN 60601-2-2 supplement 1.

**Assembly (FIG B):**

B1: Slide the insulating tube (8) onto the shaft (2a).

B2: Slide the electrode insert (7) into the shaft (2a) to the stop.

B3: Grasp the rotary knob (2b) and firmly tighten the electrode insert (7) to the stop with approximately six turns. The thread should turn easily, no canting.

⚠ Do not hold the electrode insert (7) on the (moveable) jaw part (1).

B4: Close the jaw tips, holding them closed with the fingers if necessary. For easier insertion of the shaft, press the push button (3). Hold the handle as illustrated so the moveable handle part (5) easily falls or can be brought into the rear position.

Leave the moveable handle part (5) in the rear position, without holding it.

Slide the shaft (2a) with electrode insert (7) straight into the handle. The handle part returns to the normal position, the push button engages.

It must be possible to rotate the rotary knob (2b) by 360°. The jaw part (1) must be easy to open and close when operating the handle part.

Properly connect the instrument, only to the bipolar output of the electrosurgery device.

B5: Perform a brief **functional test** prior to use:

Select medium output or 30–40 watts of power on the electrosurgery device, clamp three layers of damp mull in the jaw parts and activate the current. With correct functioning, some steam is produced after 2–3 seconds.

⚠ Do not touch the tips! The jaw parts may be hot!

**During use:**

Always work with the lowest output setting for the desired surgical effect.

⚠ Maximum allowable voltage 650 Vp

Max. output settings:

Bipolar cut, max. 100 W, maximum mean effect size

Bipolar coagulation, max. 100 W, maximum mean effect size

When using other generators, avoid spark formation in the area of the jaw tips!

⚠ Regularly wipe blood and tissue residue from the jaw tips.

⚠ Jaw tips may cause injuries!

⚠ After application, the jaw tips may be so hot they cause burns!

⚠ Never lay down instruments on or in the immediate vicinity of the patient! Lay cables and store unused instruments so they are isolated from the patient.

⚠ Not for use in the presence of combustible or explosive substances!

⚠ The uninsulated parts of the working end are live and may cause injuries!

Clean the jaw tips with sterile fluid (distilled water) as needed during use and then dry them. Do not use saline solution! Use electrolyte-free flushing fluid when flushing instruments are used simultaneously.

⚠ Unintentional activation or movement of laparoscopy electrodes outside the field of vision can lead to unwanted tissue damage.

Avoid unintentional tissue contact with live metal components of the jaw tips during activation (FIG D)

⚠ Gas embolism may occur during laparoscopy applications.

**Disassembly (FIG C):**

First disconnect the cable from the instrument.

C2: Press push button (3) and simultaneously pull the shaft (2a) completely out of the handle.

C3: Grasp the rotary knob (2b). Hold the electrode insert (7) on the front end and turn it out of the shaft (2a).

⚠ Do not hold the electrode insert (7) on the moveable jaw part (1).

C4: Pull the insulating tube (8) off the shaft (2a). Now the forceps are disassembled.

**Reconditioning:**

**General information:**

Observe national guidelines and regulations!

Disconnect the instrument from the cable!

The overall reconditioning process encompasses pre-cleaning, cleaning / disinfection, and sterilization.

Machine cleaning / disinfection is preferred due to effectiveness and reproducibility!

⚠ Do not immerse in hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>)!

**Pre-cleaning:**

- Do not allow blood and tissue residues to dry up; within no more than 1 hour, rinse thoroughly with cold water, also inside on the thread of the jaw parts and inside the tube shaft! Use a soft brush if needed (no wire brush or similar) until no more tissue residues are visible.

- Immerse the disassembled instrument in cold water for 5 minutes; the lumina must be filled.

- Rinse the lumina of the instruments for at least 10 seconds with tap water, pressure 3.8 bar.

- Moveable parts must be moved back and forth several times during pre-cleaning.

⚠ Avoid excessive manual spreading of the jaw parts! Initial damage can lead to breakage during application with repeated grasping.

**Machine cleaning and disinfection:**

In selecting the cleaning and disinfection device (CDD), note that the effectiveness of the CDD must have been tested (FDA clearance for instance, or the CE marking according to EN ISO 15883).

- Load instruments into the CDD. The lumina have to be connected to the flushing connection of the CDD. In doing so, make sure the instruments do not touch each other and are securely supported.

- The optionally available storage tray (70 17 40) ensures secure support.

Program steps	Parameters
Pre-rinse with cold tap water	2 minutes
Clean with 0.5 % (5 ml/liter) neodisher® MediClean forte	55±2 °C, 5 minutes
Neutralize with cold demineralized water	3 minutes
Final rinse with cold demineralized water	2 minutes
Thermal disinfection	Thermal disinfection according to national requirements. An A <sub>0</sub> value of 3000 can for example be obtained with a hold time of 5 minutes at 90 °C.

*Proof of fundamental suitability of the instruments for effective machine cleaning and disinfection was provided by an independent, accredited test laboratory using the Miele G7836 CD disinfectant (thermal disinfection, Miele & Cie. GmbH & Co., Gütersloh, Germany) and the cleaning agent neodisher® MediClean forte with a concentration of 0.5 % (Chemische Fabrik Dr. Weigert GmbH & Co. KG, Hamburg, Germany) test report no. 16215 dated 18.09.2015.*

- Please note: The preceding instructions contain validated minimum times for successful cleaning with the prescribed program steps. If a different cleaning agent is used, only use products with properties comparable to neodisher® MediClean forte (Dr. Weigert, Hamburg), for instance regarding the pH value and compatibility with plastics (note the instructions for the cleaner in selecting the process parameters). Please contact your supplier or hygiene officer in case of questions.

**Inspection:**

Prior to sterilization, visually inspect the instrument and check for intact insulation, cleanliness, and integrity.

**Maintenance:**

None

**Packaging:**

Package cleaned and disinfected instruments in disposable sterilization packaging (single or double packaging), or wrap the instrument or tray with the cleaned and disinfected instruments in a cotton cloth and store them together in suitable sterilization containers meeting the following requirements:

- EN ISO/ANSI AAMI ISO 11607

- Suitable for steam sterilization (resistant to temperatures up to min. 141 °C, adequate vapor permeability)

- Sufficient protection of the instruments and/or sterilization packaging against mechanical damage

**Sterilization:**

Products must be cleaned and disinfected prior to sterilization.

- Steam sterilization, steam sterilizer according to EN 13060 and/or EN 285 and validated according to EN ISO 17665

Program steps	Parameters
Process	Fractionated vacuum (3x)
Sterilization temperature	132 °C (max. 138 °C plus tolerance according to EN ISO 17665)
Sterilization time (holding time at sterilization temperature)	Min. 3 minutes
Drying time	Not specified

*Proof of fundamental suitability of the instruments for effective steam sterilization was provided by an independent, accredited test laboratory, test report no. 17315 dated 30.11.2015. Typical conditions in clinics and medical practices as well as the process described above were taken into account.*

⚠ Do not sterilize in hot air!

⚠ Do not sterilize in STERRAD®!

⚠ In case of potential contact with prions (CJD – risk of contamination), destroy the instrument and do not use it again.

#### **Storage / Transportation:**

Store in a dry place. Protect from sunlight. Store and transport in secure containers / packaging.

For returns, products must be cleaned, disinfected, and packed in sterile packaging.

Any change to the product or deviation from these instructions for use waives the liability of Sutter Medizintechnik.

Changes reserved. Current version available at: [www.sutter-med.de](http://www.sutter-med.de)

STERRAD® is a trademark of Johnson & Johnson, Inc.

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