D**E****E****E****L****P****NL****S**

Monopolare Elektroden
Monopolar Electrodes
Électrodes monopolaires
Electrodos monopolares
Electrodos monopolares
Elettrodi monopolari
Monopolaire elektroden
Monipolära elektroder

- **Standard**
- **ARROW*tip*™**
- **Arthro**
- **Micro**

REF:

36 03 20...36 03 79

36 03 42SV1

36 04 20...36 05 29

36 05 31

36 05 40...36 05 49

36 06 00...36 06 49

36 08 00...36 08 29

36 08 30...36 08 59

36 08 60...36 08 93

0297

Sutter Medizintechnik GmbH
Tullastr. 87
79108 Freiburg/Germany
Tel.: +49 (0) 761 515510
Fax: +49 (0) 761 5155130
E-Mail: info@sutter-med.de
www.sutter-med.de

DK**FIN****PL****GR****RUS****CN****(ROK)****JP**

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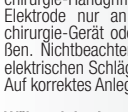
© STERRAD® is a trademark of Johnson & Johnson, Inc.

ENGLISH

Product / User / Disposal:
Electrosurgical accessories may only be used and disposed of by trained medical personnel.
These instructions do not substitute the instructions for the electrosurgical unit and other accessories used.

Non-sterile. △ Clean and sterilize before first and subsequent uses.

Intended use:
Monopolar electrode for cutting and coagulation of soft tissue during surgical interventions.

Product life:
The life cycle of the product depends on the use and treatment it gets. Replace electrode after max. 20 uses.

Prior to use:
△ Examine the integrity of the insulation as well as the cleanliness and integrity of the instrument.
△ Use a testing device to examine the integrity of long electrodes.
△ Only use safe and sterilized products.
Insert the electrode connector fully into the proper handpiece according to the instructions.
Connect instrument and cable only to an electro-surgical unit that has been switched off or is in standby mode.
△ Disregarding these instructions may lead to burns and electrical shock.
Ensure that the patient plate is properly attached.

During use:
Always use the lowest power setting possible to achieve the desired surgical effect.
△ Instrument tips may cause injuries.
△ Instrument tips may be hot after use and cause burns.
△ Never place the instrument on or close to the patient.
△ Do not use near inflammable or explosive substances
We recommend the use of non-conductive liquid when using electrodes together with irrigation instruments.
△ For maximum voltage and connector/adaptor see TAB. 1.

Cleaning and maintenance:
Observe nationally applicable regulations and guidelines.
Disconnect the instrument from the connector/adaptor/cable.
Do not allow blood and debris to dry on the instrument.
Use a soft cloth to remove blood and debris.
Do not use abrasive/abrasive cleaners.
Pre-clean and rinse thoroughly with clean water.
Give preference to machine-cleaning and thermal disinfection of the instrument.
△ Do not use hydrogen peroxide (H₂O₂) for cleaning!

Manual cleaning and disinfection:
Use cleaners suitable for plastic materials [1]. Follow the manufacturer's instructions.

Machine cleaning and disinfection:
△ Make sure that the instrument is protected against mechanical damage.
Thermal disinfection for min. 5 minutes at ≥90°C (≥194°F).
We recommend cleaners suitable for plastic materials [2].
Use an approved cleaning program. Follow the manufacturer's instructions for dosage and the program run.
Use distilled or fully desalinated water for final rinsing cycle.
Ensure that the efficacy of the disinfecter used has been validated (CE marking acc. to DIN EN ISO 15883).

Sterilisation:
Only sterilize clean and disinfected products.
Use paper/foil wrapping or appropriate sterilization containers.
Steam-sterilize in the autoclave.
Fractioned pre-vacuum sterilization: Temperature 134°C (273°F), 3 min.; max. Temperature 138°C (280°F), max. duration 20 minutes.
Ensure that the instrument is dry after sterilization!
Do not sterilize in hot air.
△ Do not sterilize in STERRAD®!

△ Destroy instruments that might have been contaminated with prions (CJD). - Do not reuse!

Storage/transport:
Store in a cool and dry place. Do not expose to direct sunlight. Store and transport in safe containers/packaging.
For returns only cleaned and disinfected instruments in sterile packaging will be accepted.

Sutter Medizintechnik cannot be held liable for changes to the product after purchase or deviations from these instructions

Subject to change.
Updates available under www.sutter-med.de.

© STERRAD® is a trademark of Johnson & Johnson, Inc.

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