Swyng® non-stick bipolar forceps, single-use

REF:

78 44 105 ST – 78 44 915 ST (Standard)

78 44 105 SL – 78 44 415 SL (Slim)

STERILE R

Rx
only

CE 0297

MD

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Tullastr. 87

79108 Freiburg/Germany

Tel.: +49 (0) 761 515510

Fax: +49 (0) 761 5155130

E-Mail: info@sutter-med.de

www.sutter-med.de

Only for US

Distributed by:

Sutter Medical Technologies USA

120 Interstate North Parkway – Suite 118

Atlanta, GA 30339 – USA

Office phone: +1 770-702-7087

Fax number: +1 470-308-3155

E-Mail: info@sutter-usa.com

www.sutter-usa.com

Content

| | | |
|-----------|---|----|
| DE | Swyng® non-stick bipolare Pinzetten, single-use _____ | 3 |
| EN | Swyng® non-stick bipolar forceps, single-use _____ | 6 |
| FR | Swyng® non-stick pinces bipolaires, single-use _____ | 9 |
| ES | Swyng® non-stick pinzas bipolares, single-use _____ | 12 |
| IT | Swyng® non-stick pinze bipolari, single-use _____ | 15 |
| NL | Swyng® non-stick bipolaire pincetten, single-use _____ | 18 |
| PT | Swyng® Pinças bipolares antiaderentes, de uso único _____ | 21 |
| SV | Swyng® bipolär non-stick-pincetter för engångsbruk _____ | 24 |
| DK | Swyng® klæbefrie bipolare pincetter til engangsbrug _____ | 27 |
| FI | Swyng® tarttumattomat bipolaariset pinsetit, kertakäyttöiset _____ | 30 |
| NO | Swyng® non-stick bipolare pinsetter, single-use _____ | 33 |
| PL | Jednorazowe, nieprzywierające pęsety bipolarne Swyng® _____ | 36 |
| BG | Swyng® биполярна пинсета с покритие против пригаряне, за еднократна употреба _____ | 39 |
| RO | Pensele bipolare neadezive Swyng®, de unică folosință _____ | 42 |
| SK | Swyng® non-stick bipolárne pinzety, single-use _____ | 45 |

| | | |
|-----------|--|----|
| SL | Swyng® bipolarna pinceta brez sprijemanja, za enkratno uporabo _____ | 48 |
| CZ | Bipolární pinzety Swyng® non-stick, single-use _____ | 51 |
| HU | Tapadásmentes bipoláris Swyng® csipesz, egyszer használatos _____ | 54 |
| LT | „Swyng® non-stick“ vienkartinis bipoliarinis pincetas _____ | 57 |
| LV | Swyng® nepielīpošas, vienreizlietojamas bipolārās pincetes _____ | 60 |
| EL | Διπολική αντικολλητική λαβίδα Swyng®, μιας χρήσης _____ | 63 |
| HR | Bipolarna non-stick pinceta Swyng®, jednokratna uporaba _____ | 66 |
| SR | Swyng® non-stick bipolarne pincete za jednokratnu upotrebu _____ | 69 |
| MK | Swyng® нелеплива биполарна пинцета за еднократна употреба _____ | 72 |
| TR | Swyng® non-stick single-use bipolar pensler _____ | 76 |
| ZH | Swyng® 一次性非粘性双极镊 _____ | 79 |
| AR | ثنائي القطب، غير اللاصق، للاستخدام لمرة واحدة Swyng® ملقط EG/JO/KE/LB/MA/AE _____ | 82 |
| HE | כולל ציפוי נגד הדבקה, Swyng®, פינצטה דו-קוטבית חד-פעמית _____ | 85 |
| EN | Swyng® non-stick bipolar forceps, single-use US/SA _ | 88 |
| AR | غير لاصق، أحادي الاستخدام Swyng® ملقط قطيبي SA مُقتبس من إصدار أمريكي _____ | 92 |

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Product/user/disposal:

Electrosurgery accessories may only be used and disposed of by qualified medical personnel!

Notwithstanding these instructions, reading the instructions for use provided with the electrosurgery device and other accessories being used is mandatory. For further information about electrical safety, we recommend IEC TR 61289 or DIN EN 60601-2-2 supplement 1.

Sterile product:

⚠ Do **NOT** recondition the instrument.

The thermolabile plastics used have **not** been validated regarding potentially suitable cleaning and sterilization processes.

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Intended purpose:

Swyng® non-stick bipolar forceps single-use are used for the coagulation of tissue during surgery.

Contraindications

- Resterilization of the used product by the customer.

Service life:

For single use only.

Before use:

⚠ Prior to use, inspect the product for:

- Sound condition of the sterile packaging.
- Integrity of the instrument.
- Remove the protective tubing from the tips of the forceps.
- To connect the forceps, the electrosurgery device must be turned off or in standby mode.
- ⚠ Failure to comply with this instruction can result in burns and electric shocks!

⚠ Do **NOT** use the instrument in the following cases:

- Discernible damage to the sterile packaging or forceps.
- The package has gotten damp or was opened.
- The sterility indicator does not have the required color.
- The expiration date has been exceeded.

Always have sterile bipolar forceps on hand as a replacement!

Electrosurgery cables:

The Sutter bipolar forceps single-use are intended for use with bipolar cables with a US connector from the manufacturer Sutter Medizintechnik GmbH. If cables from other manufacturers are used, please ensure that the cable is fitted securely to the instrument and to the generator. The cable being used must be designed for a voltage of at least 500 Vp.

During use:

⚠ Always work with the lowest output setting for the desired surgical effect.

⚠ Regularly wipe off blood and tissue residues from the tips.

⚠ Never lay down the instrument on or in the immediate vicinity of the patient during surgery.

DO NOT put any heavy objects on top. Lay cables and store unused instruments so they are isolated from the patient.

⚠ Tips of the forceps can cause injuries!

Directly after turning off the RF current, the tips of the forceps may be so hot they cause burns.

⚠ Not for use in the presence of combustible or explosive substances!

⚠ Max. voltage 500 Vp

Storage/transportation:

⚠ Note the sterility expiry date!

Store in a dry place. Protect from sunlight. Store at 10 °C to 30 °C, relative humidity 30 - 80 %, non-condensing.

Please note:

Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the com-

petent authority of the Member State in which the user and/or patient is established.

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Changes reserved.

Current version available at
www.sutter-med.de.

Explanation of symbols:

| | |
|---|---|
|  | Don't use if packaging is damaged |
|  | Single sterile barrier system |
|  | Single sterile barrier system with protective packaging outside |
|  | Medical device |

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Description:

Swyng® non-stick bipolar forceps, single-use are available in a variety of lengths and tip sizes for use with a common bipolar electrosurgical generator. They are to be connected through an appropriate bipolar cable with the bipolar output of an electrosurgical generator. The forceps are for single use only and are provided sterile.

Indication for use:

Swyng® non-stick bipolar forceps, single-use are intended for use in electrosurgery for coagulation of tissue.

Contraindications:

- Resterilization of used product by the customer.
- Use for contraceptive coagulation of fallopian tube tissue.

Product / user / disposal:

Caution: Electrosurgery accessories may only be used and disposed of by qualified medical personnel!

Notwithstanding these instructions, reading the instructions for use provided with the electrosurgery device and other accessories being used is mandatory.

Sterile product:

Caution: Do **NOT** reprocess the instrument.

The thermolabile plastics used have not been validated regarding potentially suitable cleaning and sterilization processes.

Dispose of the instrument according to internal hospital guidelines for sharp, biologically contaminated items.

Service life:

For single use only.

Before use:

Warning: Prior to use, inspect the product for:

- Intactness of the sterile packaging
- Integrity of the instrument

To connect the forceps, the electrosurgery device must be in standby mode.

Connect the forceps with an appropriate bipolar cable to the bipolar output of an electrosurgical generator.

Warning: Failure to comply may lead to burns and electric shocks!

Do **NOT** use the instrument in the following cases:

- Damage to the sterile packaging or forceps
- The package has gotten damp or was opened
- The sterility indicator does not have the required color
- The expiration date has been exceeded

Always have a sterile bipolar forceps on hand as a replacement!

During use:

- Do **NOT** exceed 500 Vp in the bipolar mode. Failure to observe mode and setting warnings may result in unintended burns and/or cause failure of the device insulations. Notwithstanding these instructions, reading the instructions for use provided with the electrosurgical generator and other accessories being used is mandatory.
- Always work with the lowest output setting for the desired surgical effect.
- Regularly wipe blood and tissue residue from the tips.
- Never lay down the instrument on or in the immediate vicinity of the patient during surgery. Do **NOT** put any heavy objects on top. Lay out cables and store unused instruments so they are isolated from the patient.
- Tips of the forceps may cause injuries!

- Do **NOT** use in patients who have electronic implants such as cardiac pacemakers without first consulting a qualified professional (e.g., cardiologist). A possible hazard exists because interference with the action of the electronic implant may occur, or the implant may be damaged.
- Do **NOT** use in the presence of flammable anesthetics or oxidizing gases (such as nitrous oxide (N₂O) and oxygen) or in close proximity to volatile solvents (such as ether or alcohol), as explosion may occur.
- Do **NOT** place instruments near or in contact with flammable materials (such as gauze or surgical drapes). Instruments that are activated or hot from use may cause a fire.
- The surface of the active electrode may remain hot enough to cause burns after the RF current is deactivated.
- Due to concerns about the carcinogenic and infectious potential of electrosurgical byproducts (such as tissue smoke plume and aerosols), protective eyewear, filtration masks, and effective smoke evacuation equipment should be used in both open and laparoscopic procedures.
- Do **NOT** activate the instrument when not in contact with target tissue, as this may cause injuries due to capacitive coupling with other surgical equipment.

Storage / transportation:

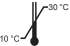
Caution: Note the sterility expiry date!

Store in a cool, dry place. Protect from sunlight. Store at 10 °C (50 °F) to 30 °C (86 °F), relative humidity 30 – 80 %, non-condensing.

Sutter Medizintechnik GmbH expressly warns against modifying the instrument. Any change or deviation from the instructions provided here waives the liability of Sutter Medizintechnik GmbH.

Changes reserved.

Explanation of symbols:

| | |
|---|--|
|  | See instructions for use |
|  | Protect from humidity |
|  | Protect from sunlight |
|  | Single use only – not for reuse |
| LOT | Batch code |
| REF | Article number |
| STERILE R | Sterilized using irradiation. Sterility guaranteed unless package opened or damaged. |
|  | Temperature limitation |
|  | Manufacturer |
|  | Date of manufacture |
| Rx only | Caution: Federal (USA) law restricts this device to sale by or on the order of a physician |
| CE ₀₂₉₇ | European conformity symbol |
|  | Date of expiry |
|  | Do not use if package is damaged |
|  | Single sterile barrier system |
|  | Single sterile barrier system with protective packaging outside |
| MD | Medical device |

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