# Monopolar Electrodes ARROWtip™ Single-Use

REF: 36 44 20 - 36 44 75

STERILE EO

**Rx**  
only

**CE** 0297

**MD**

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Tullastr. 87

79108 Freiburg/Germany

Tel.: +49 (0) 761 515510

Fax: +49 (0) 761 5155130

E-Mail: [info@sutter-med.de](mailto:info@sutter-med.de)

[www.sutter-med.de](http://www.sutter-med.de)

Only for US

Distributed by:

Sutter Medical Technologies USA

120 Interstate North Parkway – Suite 118

Atlanta, GA 30339 – USA

Office phone: +1 770-702-7087

Fax number: +1 470-308-3155

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**Product/user/disposal:**

Electrosurgery accessories may only be used and disposed of by qualified medical personnel!

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**Sterile product:**

⚠ Do **NOT** recondition the instrument.

The thermolabile plastics used have **not** been validated regarding potentially suitable cleaning and sterilization processes.

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**Intended use:**

The monopolar electrodes are intended for cutting and coagulating soft tissue during surgical procedures.

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The monopolar electrodes have a standard 2.4 mm (3/32") diameter connection and are compatible with the monopolar handles and RF generators common in the market. We recommend using monopolar handles from Sutter Medizintechnik GmbH.

**Service life:**

For single use only.

**Before using:**

⚠ Prior to use, inspect the product for:

- Sound condition of the sterile packaging
- Integrity of the instrument
- To connect the electrode, the electrosurgery device must be turned off or in standby mode
- Insert the connection end of the electrode fully and properly into the electrosurgery handle intended for the purpose
- Pull off the protective cap on the tip of the electrode

⚠ Failure to comply may lead to burns and electric shocks!

Verify that the neutral electrode is applied correctly! For further information about electrical safety, we recommend DIN EN 60601-2-2 supplement 1.

⚠ Do **NOT** use the instrument in the following cases:

- Discernible damage to the sterile packaging or electrode
- The package has gotten damp or was opened
- The sterility indicator does not have the required color
- The expiration date has been exceeded

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### **During use:**

- Always work with the lowest output setting for the desired surgical effect.
- Never lay down the instrument on or in the immediate vicinity of the patient during surgery. Do NOT put any heavy objects on top. Lay cables and store unused instruments so they are isolated from the patient.

⚠ Electrode tips may cause injuries!

⚠ Directly after turning off the RF current, electrode tips may be so hot they cause burns.

⚠ Unintentional activation or movement of the activated electrode outside the field of vision can lead to unidentified tissue damage.

⚠ Not for use in the presence of combustible or explosive substances!

⚠ When flushing instruments are used simultaneously with the electrode, use non-conductive flushing fluid if possible!

⚠ Max. voltage 3000 Vp

### **Storage/transportation:**

⚠ Note the sterility expiry date!

Store in a dry place. Protect from sunlight. Store at 10 °C to 30 °C, relative humidity 30 – 80 %, non-condensing.

### **Please note:**

Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Current version available at [www.sutter-med.de](http://www.sutter-med.de).

Explanation of symbols:

	Do not use if packaging is damaged
	Single sterile barrier system
	Medical device

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**Product / User / Disposal:**

**Caution:** Electrosurgery accessories may only be used and disposed of by qualified medical personnel!

Notwithstanding these instructions, reading the instructions for use provided with the electrosurgery device and other accessories being used is mandatory.

**Sterile product:**

**Caution:** Do **NOT** reprocess the instrument.

The thermolabile plastics used have not been validated regarding potentially suitable cleaning and sterilization processes.

Dispose of the instrument according to internal hospital guidelines for sharp, biologically contaminated items.

**Intended use:**

The ARROWtip™ Single-Use monopolar electrode is an electrosurgical instrument for precision soft tissue dissection. It is a single-use device intended for cutting, dissecting and cauterizing soft tissue. The ARROWtip™ Single-Use monopolar electrode is not intended for use in the central nervous system or in the central circulatory system.

**Electrosurgery cables and handles:**

The monopolar electrodes have a standard 2.4 mm (3/32") diameter connection and are compatible with monopolar handpieces and RF generators common in the market.

**Service life:**

For single use only.

**Before use:**

**Warning:** Prior to use, inspect the product for:

- Sound condition of the sterile packaging
- Integrity of the instrument
- To connect the electrode, the electrosurgery device must be turned off or in standby mode
- Insert the connection end of the electrode fully and properly into the electrosurgery handpiece

- Pull the protective cap off the end of the electrode

**Warning:** Failure to comply may lead to burns and electric shocks!  
Verify that the neutral electrode is applied correctly!

Do **NOT** use the instrument in the following cases:

- Damage to the sterile packaging or electrode
- The package has gotten damp or was opened
- The sterility indicator does not have the required color
- The expiration date has been exceeded

Always have a sterile monopolar electrode on hand as a replacement!

### **During use:**

- Always work with the lowest output setting for the desired surgical effect.
- Never lay down the instrument on or in the immediate vicinity of the patient during surgery. Do NOT put any heavy objects on top. Lay out cables and store unused instruments so they are isolated from the patient.
- **Warning:** Electrode tips are sharp and may cause injuries!
- **Warning:** Directly after turning off the RF current, electrode tips may be so hot they cause burns.
- **Warning:** Unintentional activation or movement of the activated electrode outside the field of vision can lead to unidentified tissue damage.
- **Warning:** Not for use in the presence of combustible or explosive substances!
- When irrigating instruments are used simultaneously with the electrode, use non-conductive irrigation fluid if possible!
- **Caution:** Max. voltage 3000 Vp

### **Storage / Transportation:**

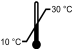
**Caution:** Note the sterility expiry date!

Store in dry place. Protect from sunlight. Store at 10 °C to 30 °C, relative humidity 30-80 %, non-condensing.

Sutter Medizintechnik GmbH expressly warns against modifying the instrument. Any change or deviation from the instructions provided here waives the liability of Sutter Medizintechnik GmbH.

Changes reserved. Current version available at [www.sutter-usa.com](http://www.sutter-usa.com).

Explanation of symbols:

	Observe instructions
	Protect against moisture
	Protect against sunlight
	Single use only – not for reuse
<b>LOT</b>	Batch code
<b>REF</b>	Article number
<b>STERILE EO</b>	Sterilized with ethylene oxide. Sterile as long as the packaging is unopened and intact
	Temperature limit
	Manufacturer
	Date of manufacture
<b>Rx</b> only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
<b>CE</b> 0297	European conformity symbol
	May be used until
	Do not use if packaging is damaged
	Single sterile barrier system
<b>MD</b>	Medical device

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