

# SuperGliss® non-stick Bipolar Forceps

Please pass to sterile department!



If necessary, use a soft brush or cloth to remove any residue. **Do not use** a wire brush, scratch sponge, scalpel, or similar items.



Do not bend the forceps.



The material of the tips can darken through contact with cleaners and disinfectants, this oxide layer is biocompatible and harmless.



Always protect the tips and tines of the forceps, as they are softer than steel.



If necessary, please use the optionally available storage trays to protect the instrument.



Minor color changes of the red coating are usually harmless.



The overall reconditioning process encompasses pre-cleaning, cleaning/disinfection, and sterilization. Please refer to the instrument's instructions for use for more information and observe national guidelines and regulations.

© Sutter Medizintechnik GmbH · Subject to change · REF 1431 A - V02



PRECISION ELECTROSURGERY  
Made in Germany

SUTTER MEDIZINTECHNIK GMBH  
ALFRED-WALZ-STR. 22 · 79312 EMMENDINGEN/GERMANY  
TEL. +49(0)7641-96256-0 · FAX +49(0)7641-96256-30  
WWW.SUTTER-MED.COM · INFO@SUTTER-MED.DE