Starre bipolare Elektroden
Rigid Bipolar Electrodes
Électrodes bipolaires rigides
Electrodos bipolares rígidos
Electrodos bipolares rígidos
Electrodi bipolari rigidi
Onbuigzame bipolaire elektroden
Bipolära elektroder

REF:
70 04 50 - 70 04 69
70 04 95 - 70 04 99
72 01 16 - 72 01 40

Sutter Medizintechnik GmbH
Tullastr. 87
79108 Freiburg/Germany
Tel.: +49 (0) 761 515510
Fax: +49 (0) 761 5155130
E-Mail: info@sutter-med.de
www.sutter-med.de
www.sutter-med.com

DOC_REF 89 90 02_02
Stand 04/2008
© Sutter

A

B

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DEUTSCH

ENGLISH

Non sterile. – Clean and sterilize before first and subsequent uses.

Electrosurgical accessories may only be used and disposed of by trained medical staff!
These instructions do not substitute the reading of the instructions for the electrosurgical unit and other accessories used.

Intended use:
Electrodes REF 70 04 50 – 70 04 99 are intended for bipolar coagulation of soft tissue.
Electrodes REF 72 01 16 – 72 01 40 are intended for bipolar cutting and coagulation of soft tissue in laparoscopy.

Prior to use:
Examine the integrity of the insulation as well as the cleanness and integrity of the instrument.

⚠ Only use safe and sterilized products!
Connect the instrument properly to the bipolar outlet of the electrosurgical unit.
⚠ Connect the instrument only to an electrosurgical unit that has been switched off or is in standby mode. Disregarding these instructions may lead to burns and electrical shock.

During use:
Always use the lowest possible power setting to obtain the desired surgical effect.
Max. voltage: 1000 Vpp.
⚠ Electrode tips may cause injuries.
After activation electrode tips may be hot and cause burns!
Never place an electrode on or close to the patient.
Do not use with inflammable or explosive substances.
Undesired activation or movement of laparoscopic electrodes outside the surgeon's field of vision may lead to undesired tissue damage.
Gas embolisms may occur in laparoscopic surgery.
Not intended for female sterilization.

Cleaning and sterilization:
⚠ Always observe nationally applicable regulations and guidelines.
Disconnect cable from the electrode.
Do not allow blood and debris to dry on the electrode.
Manual cleaning:
Use a soft cloth or brush to remove blood and debris.
=> Do not use aggressive/abrasive cleaners.
Rinse thoroughly with clean tap water after cleaning.
Machine cleaning/disinfection:
Use an approved cleaning programme (93 °C (200 °F), > 10 min).
We recommend enzymatic cleaner. Follow the manufacturer's dosage instructions.
Cleaning in ultrasound possible. Ultrasound may shorten the life of the instrument.
=> Perform visual control prior to sterilization and verify the integrity of the insulation as well as cleanness and integrity of the entire instrument.
Only sterilize clean/disinfected instruments.
Use paper/foil wrapping or appropriate sterilization containers.
Sterilization in the autoclave (fractioned prevacuum):
121 °C (250 °F), 20 min or 134 °C (273 °F), 5 min.
Max. temperature 138 °C (280 °F), max. sterilization time 18 min.
Do not sterilize in hot air.
=> Sterilization at high temperatures and long sterilization times shorten the life cycle of the product.

⚠ Sutter cannot be held liable for changes to the product after purchase or deviations from these instructions.

Information on applicable standards, regulations and existing validations may be obtained from the manufacturer.

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