
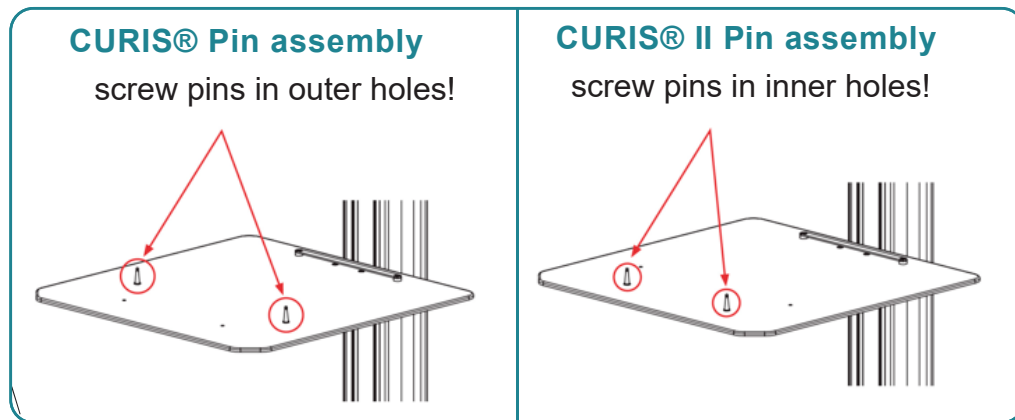
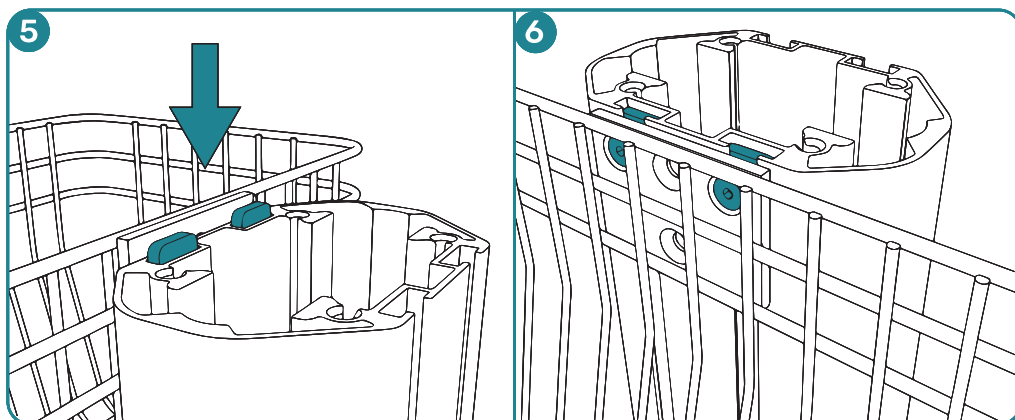
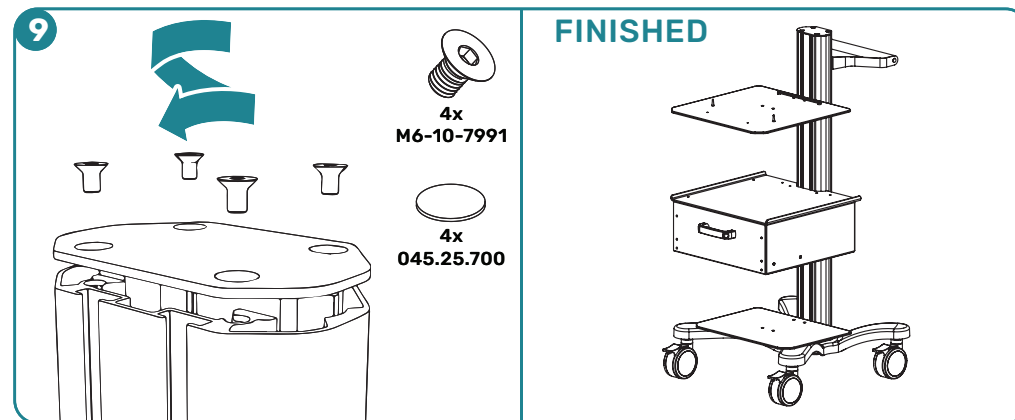
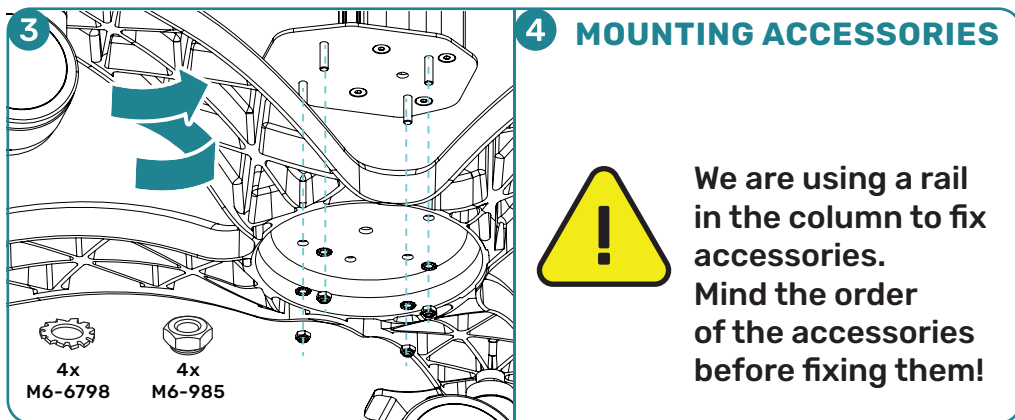
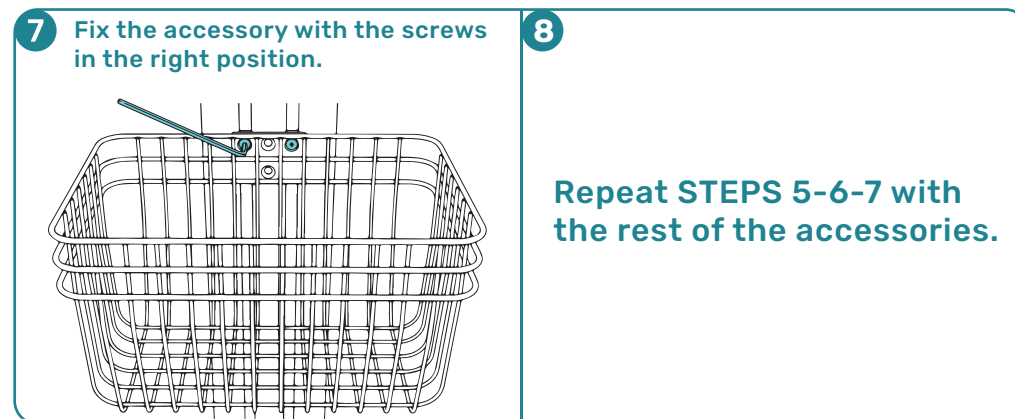


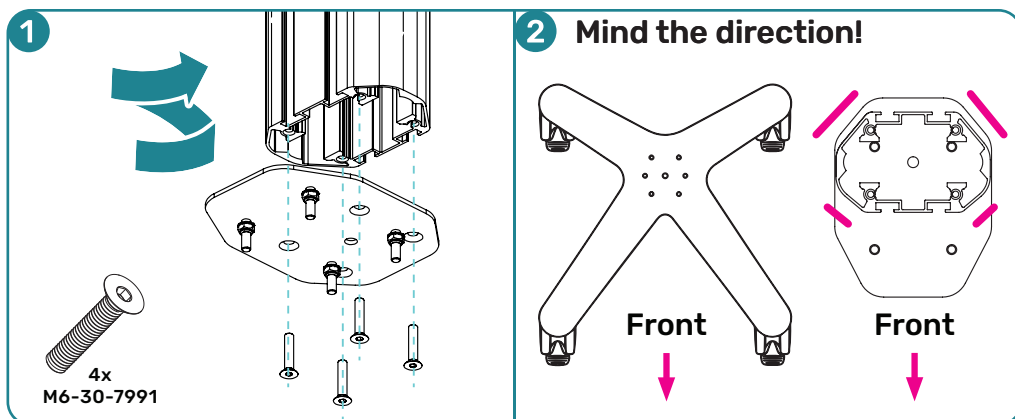

! Cleaning procedure

1. The Cart may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment such as diluted bleach, ammonia, or alcohol solutions.
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylen.
3. Do not use steel wool or other abrasive material to clean the cart.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommended that you test any cleaning solution on a small area of the cart that is not visible to verify compatibility.
5. Wipe any cleaning agents off of the cart immediately using a water-dampened cloth.
6. Dry the cart thoroughly after cleaning.

CAUTION: Medical Cart Company makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

! Installation and maintenance warnings

- This section contains warnings regarding the assembly and maintenance of the Cart. This section must be read in its entirety before installing and maintaining the Cart. Failure to follow these warnings may result in damage to equipment or injury to personnel.
- Medical Cart Company has tip tested the complete cart to verify stability. Configurations differing from approved Medical Cart Company specifications should be tip tested to ensure safe operation and prevent any stability/tip hazards. Contact Medical Cart Company for further information.
- Please consider carefully the mounted equipment and proximity of cart to other equipment, hospital personnel and the patient.
- When moving the Cart fold any extended mounts with equipment inward to prevent collisions with other equipment, walls, doorways, hospital personnel and the patient.
- Periodically inspect the Cart components and fasteners for looseness.
- Tighten as required for optimal operation and safety.

Allowable Cleaning Solutions

General purpose detergent
Alcohol (70% isopronanol)
Chlorine (10,000 PPM)



Do not sit on the product





Use designated handle only for moving the cart



Read Instructions!

Cart weight

$$\begin{matrix} \text{24kg} & + & \text{20kg} & = & \text{44kg} \\ \text{Cart} & & \text{Max.} & & \text{System} \\ \text{Weight} & & \text{Loading} & & \text{Gross} \\ & & & & \text{Weight} \end{matrix}$$

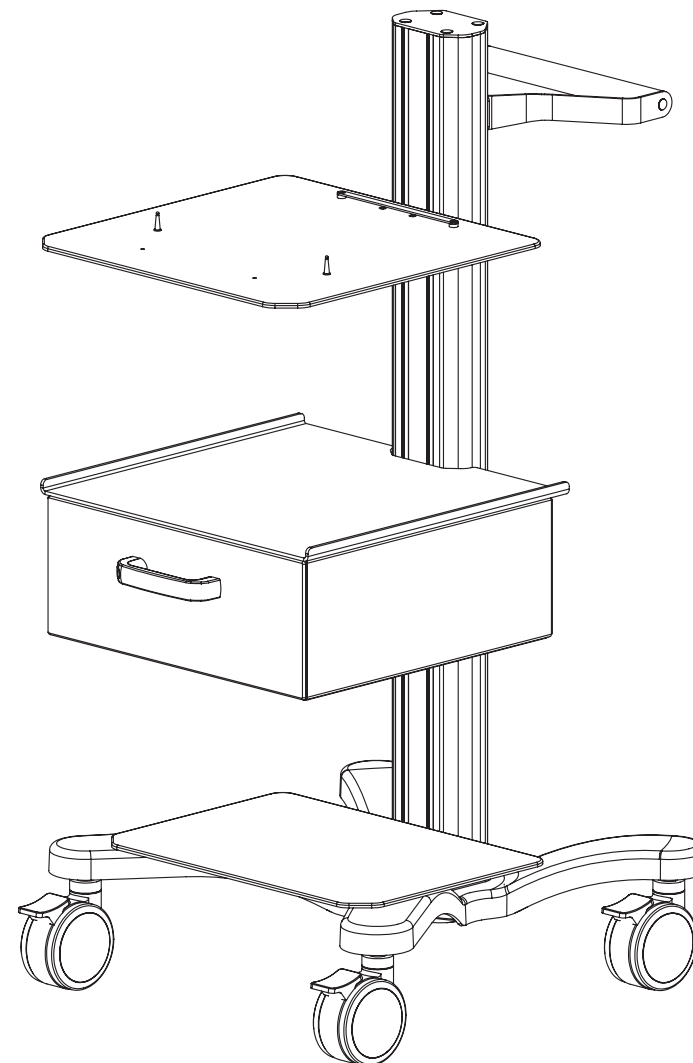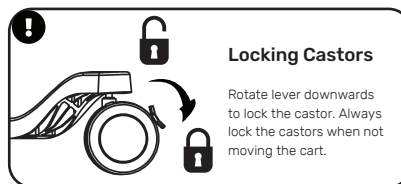

Manufactured by:

Medical Cart Company Kft.
2051 Biatorbágy, Hungary
Budai út 16.

Medical Cart
COMPANY

